EN English

WaterWear for BTE

by smartraxna

WaterWear for BTE (hereafter referred to as WaterWear) is not an accessory for the SONNET series or OPUS 2 audio processor (hereafter referred to as audio processor), but provides optional dust and water protection according to class IP68 (tested for 16 hours at a water depth of 4 meters [13 ft]). Please observe the limitations and conditions stated below when using WaterWear.

Each WaterWear may be used up to **20 times**. Do not wear or immerse in water for more than **16 hours** at a time. Use a new adhesive strip for each application.

Please read these instructions carefully and follow the recommendations. Improper use may cause humidity or dust to migrate into WaterWear and damage the audio processor.

Instructions for use
Clean and thoroughly dry your hands before handling WaterWear. You should not use hand cream or lotion. Make sure the inside of WaterWear is dry and clean and check for damage. **Never use a damaged WaterWear!** Switch on the audio processor.

1. Insert the coil into WaterWear, sliding it carefully into the flexible sleeve without damaging the cable. Make sure that the coil cable is fully inserted in the sleeve and remains attached to the coil. **Important:** WaterWear can be worn on the left and right side of the head. Do not worry about correct orientation when inserting the coil as the coil can easily be turned inside WaterWear.

2. Hold the battery pack between thumb and forefinger and insert the earhook into WaterWear until it sits tightly in the tip.

3. Hold the control unit with one hand. With the other hand hold WaterWear between thumb and forefinger and pull it over the battery pack. WaterWear will fit tightly, so you will feel resistance when pulling it over the battery pack. Do not pull at the semi-circular tab (additional fixation option, easily recognizable by its fixation hole).

4. Pull WaterWear over the entire audio processor. Avoid crinkling the edges of the opening. To avoid tension on the coil cable, make sure that the coil cable is not twisted inside WaterWear.

5. Before attaching the adhesive strip, make sure that the strip and the gluing surfaces of the audio processor and WaterWear are dry and dust-free. Remove the adhesive strip from the protective cardboard and stick it to the WaterWear opening. Start applying the adhesive strip at the bottom of the battery pack but do not yet seal it completely. Place the fingers of your free hand over the coil cable to press out any remaining air. Make sure that there is no excess air in WaterWear to avoid bulging. Press down firmly along the adhesive section. To provide protection against water and dust, the adhesive strip must completely adhere to all edges of the WaterWear opening. The adhesive strip must **fully cover the reinforcement foil at the opening of WaterWear and must not be offset or misaligned. The adhesive strip is considered misaligned or not properly applied if the complete overlap between adhesive strip and reinforcement foil is less than 5 mm (0.2 in.) on each side of the reinforcement foil.** Check again if WaterWear is still undamaged.

6. Position the audio processor with WaterWear behind your ear over the implant. To avoid losing the processor, we **strongly recommend using additional fixation** (e.g. bathing cap).

7. To remove the audio processor from WaterWear, carefully pull at the additional flap of the adhesive strip. Remove and discard the adhesive strip.

8. Use your thumb and forefinger to hold WaterWear and pull it over the end of the battery pack to remove the audio processor.

Warnings and limitations

- Cochlear implant users of all ages may use WaterWear. WaterWear shall only be applied and removed by persons over 18 years to ensure correct application and to avoid swallowing WaterWear and/or the adhesive strip. Children shall use WaterWear only under adult supervision.
- Keep WaterWear and the adhesive strips out of the reach of children.
- WaterWear must not be used if the user is known to be intolerant of the materials used in WaterWear (WaterWear: Thermoplastic Polyurethane Elastomere TPE-U, Adhesive Strip: Polymeric Vinyl).
- Acoustic performance of the cochlear implant may be slightly reduced by WaterWear.
- WaterWear will protect the audio processor only when used properly. Any deviation from the described use may damage the audio processor.
- When opting to use the audio processor with WaterWear, use only alkaline, silver-oxide or rechargeable batteries¹ recommended by MED-EL instead of zinc-air batteries. Zinc-air batteries require air supply which cannot be provided when using WaterWear.
- The design of WaterWear is optimized for use with coil cable lengths of up to 9.5 cm (3.7 in.). Longer coil cables may prevent correct application of WaterWear and cause ingress of water and/or dust.

- WaterWear can only be used with the standard battery pack covers. Do not use WaterWear with smaller battery pack covers (e.g. OPUS 2 XS or SONNET battery pack cover Micro).
- WaterWear cannot be used with the FM battery pack cover. Use the standard battery pack cover instead of the FM battery pack cover.

- If you have a SONNET series audio processor, we recommend using the microphone in omnidirectional mode. With Natural or Adaptive directionality, directional hearing may be optimized by WaterWear.
- If you have a SONNET series EAS audio processor, you should replace the SONNET EAS earhook with the standard earhook (see the applicable audio processor user manual). If you are using an earmould with your audio processor, you should remove the earmould. Otherwise WaterWear cannot be applied.
- The environmental conditions of the audio processor apply when using the audio processor with WaterWear (see your audio processor user manual). The operating temperature range is 0°C (+32°F) to +50°C (+122°F). Normally, when the audio processor is worn on the body, natural body heat helps maintain this temperature range. Do not leave the audio processor with WaterWear assembled in direct sunlight (particularly inside a car). The adhesive strip should only be attached within a temperature range of +10°C (+50°F) to +50°C (+122°F). Do not assemble WaterWear under degrading conditions.

- Do not store WaterWear in direct sunlight as this can degrade the material and potentially impair its function. MED-EL recommends storing WaterWear in its protective packaging until use.
- The shelf life of the adhesive strip is 5 years.
- WaterWear should only be used when protection against excessive humidity is desired (e.g. when swimming).

- WaterWear is designed to withstand common substances associated with sports activities and swimming (like chlorinated or salt water) and against common cosmetic substances (like mild soap, sunscreen or similar). It is not designed to withstand chemicals in general (i.e. acetone or 99% ethanol). Such substances can damage and destroy WaterWear.
- To prevent losing the audio processor, additional fixation (e.g. bathing cap) is strongly recommended. Avoid subjecting WaterWear directly to strong water currents.
- Using fixations or sharp objects and materials which can scratch, cut or tear WaterWear may also damage the audio processor. Avoid damaging WaterWear with your fingernails.
- Follow the instructions below to remove the audio processor from WaterWear. Do not use sharp objects as this may damage the audio processor.
- Condensation can build up inside WaterWear when swimming. This is normal and due to the water being cooler than the air inside the cover. If condensation occurs, remove your audio processor and dry it off with a soft cloth after swimming.
- After using the audio processor with WaterWear it is recommended to dry the audio processor with the supplied drying kit (see your audio processor user manual).
- To protect the environment, make sure to properly dispose of WaterWear after use.

MED-EL's warranty for WaterWear is in compliance with and limited by compulsory statutory warranty provisions. Warranty shall be granted for material and manufacturing flaws; it shall not apply for damages caused by accidents, negligence, normal wear, inappropriate care or non-compliance with the warnings and instructions for use stated in this document. This warranty does not cover the audio processor stored in WaterWear. MED-EL shall not assume any liability for repair costs, replacement of the audio processor or any indirect consequential damage from use of WaterWear. If you experience problems with your audio processor after using WaterWear, please include the respective WaterWear with the audio processor that is returned for evaluation.

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